



BergaMASK®

Made in Italy

Maschera Protettiva
CE 2001/95



Content:

- ✓ N°1 Silicon mask
- ✓ N°1 Plastic insert for locking the filter
- ✓ N°1 pack of 30 GENERIC Filters (CE 2001/95)

Specifications:

- ✓ Self-sterilizing in microwave or any heat source
- ✓ Certified hypoallergenic and antibacterial medical silicone
- ✓ Ergonomic and adjustable
- ✓ Easy to assemble disposable filters
- ✓ Filtering capacity guaranteed for 8/10 hours of continuous use
- ✓ Filtering capacity tested on the complete assembly (mask + filter)
- ✓ The silicone mask can house all types of BergaMASK filters (generic, FFP2, FFP3, surgical IIR)

Certifications:

- ✓ 2001/95/CE
- ✓ IEC 61882:2016
- ✓ IEC 61511-1:2016 Functional safety
- ✓ FDA EU 2002/72/CE
- ✓ RoHs, REACH

Instructions for use:

- ✓ Insert the TNT filter inside the special central seat and lock it by inserting the plastic insert
- ✓ Insert the cords inside the appropriate seats
- ✓ Put on the mask by bringing the elastic bands behind the neck or behind the ears and adjusting them to your comfort

Sterilization method:

- ✓ Self-sterilizing mask in microwave for 5 minutes at 600w, oven at 100 ° C, autoclave (max 200 ° C), boiling water
- ✓ Plastic insert that can be sterilized in the oven at 100 ° C or in boiling water

